MANUALE D'USO SLEEPY

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USER MANUAL SLEEPY

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l..
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l. declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO S.r.l. in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l. declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO S.r.l. comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO S.r.l. guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

SAFETY

- Do not assemble this product before reading this user manual.
- Any modification of the product without the knowledge of the manufacturer is the full responsibility of the person making the modifications. Modifications made without authorization will void any warranties and invalidate the declaration of compliance with applicable directives.
- It is forbidden to use the product in any way other than that described in this manual.
- Demarta Virginio S.r.l. assumes no responsibility for malfunctions or damage to things and people due to improper use of the product.

- Operators should carefully read this manual, follow the instructions provided and familiarize themselves with the procedures for using and maintaining this product.
- When the patient is not under surveillance, the bed should be placed in the lowest position with the sides raised to reduce the risk of falls and injuries.
- Avoid any inappropriate use of the power cord. The resulting dangers could be, for example, bends, cuts and other mechanical damage.
- Use and perform any maintenance on the product following the instructions provided in this manual and use only spare parts and assistance approved by Demarta Virginio SAS.
- Do not use the product for purposes other than those for which it was built and designed.
- Keep this manual for advice or training of personnel.

Warning! Do not exceed the maximum range.

Warning! Always keep the bed on a level surface.

Warning! Do not attempt to cross passages, steps or other obstacles.

Warning! Do not place limbs near moving parts.

- The bed must be equipped with adequate equipment in order to keep the power cord free of any moving parts or mechanisms of the medical bed when it is in operation, or during movement, in order to avoid damage to the power cord.
- Power cords must be adequately protected from damage that may result from contact with one or more moving parts or from rubbing against sharp corners and surfaces inside the bed.

DESCRIPTION

- Two models available: standard (code 01.10.LT10) and bariatric (code 01.10.LT10-B).
- Wooden headboard.
- Wooden footboard.
- Integrated wood-effect aluminum sides.
- Steel frame divided into 4 sections.
- Painted steel structure.
- Welded slatted base.
- Variable height with electric control: from 38 cm to 79 cm.
- Size of the support surface: standard model 90x200 cm, bariatric model 120x200 cm.
- Remote control with cable.
- Possibility of Trendelenburg movement up to 12 ° and anti-Trendelenburg up to 12 ° by means of a command on the remotecontrol.
- Combined lying surface motor with built-in control unit.
- Back raising from 0 ° to 70 °.
- Leg inclination with electric control from 0 ° to 25 °.
- Support to get out of bed included.
- Four swivel castors with Ø 125 mm brakes.

Electric components

Supply voltage 230V AC

Operating voltage 24V DC

MAX power consumption 270 W

MAX actuators thrust force 6000 N

MAX hand control voltage 24V DC

TECHICAL FEATURES


Code	Total Length	Internal Length	Total Width	Internal Width	Height	Wheel Dimension	Maximum Load	Weight
Measures in Cm								
01.10.LT10	215	200	102	90	38/79	12.5 (5")	170	80
01.10.LT10-B	215	200	132	120	38/79	12.5 (5")	280	100

ASSEMBLY

The following are the warnings on the use of the medical device.

Before operating the moving parts of the bed, make sure that no pipes, electric cables ... are damaged.

The wheels have the sole purpose of moving the bed when necessary. And they must be kept braked at any other time.

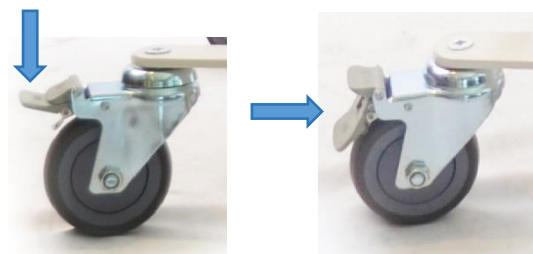Warning! Do not use the bed if you notice any anomalies related to incorrect assembly.

Warning! Do not use the bed in environments subject to splashing water or critical and adverse weather conditions.

Warnings!

The manufacturer is relieved of any responsibility for damages resulting from misuse by untrained personnel, unauthorized alterations or operations, use of non-original spare parts, exceptional events, total or partial disregard of the instructions displayed in this manual.

BOX 1 PIECES:

Bed slats:


Part 1: Backrest (A)

Part 2: Pelvis + Feet (B)

Control unit (C) and two locking supports (D)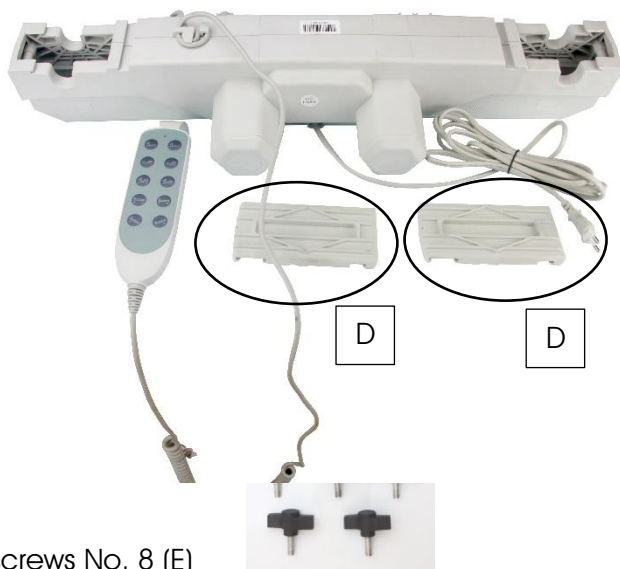


Star screws No. 8 (E)

Remote control (F):

1. Raise / lower backrest
2. Raise / lower footboard
3. Raise / lower backrest + footboard simultaneously
4. Raise / lower the bed frame
5. Trendelenburg / reverse Trendelenburg

IMPORTANT!!

The remote control has a key lock system at the rear.
To lock the keys: closed padlock logo
To unlock the keys: open padlock logo

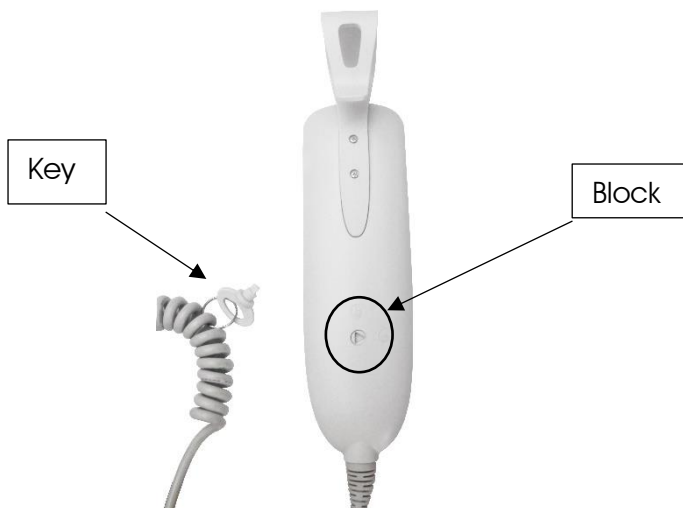Headboard (G)


Footboard (H), Demarta logo on the corner

BOX 2 PIECES:

Side nr 4 (I)

Guide n. 4 (L)

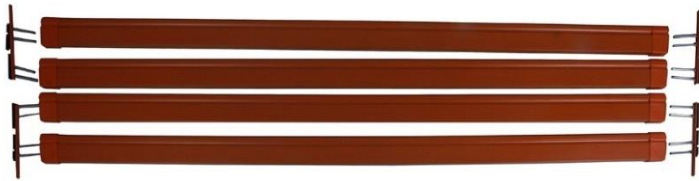Black screws 4 (M)

BOX 3 PIECES:

U support (N) and handle (O)

IMPORTANT! At least two operators are required to assemble the bed.

Base assembly of the bed:

Hook the headboard (G) to part 1 (A) (backrest) and fix with 2 star screws (E).

Hook the footboard (Demarta logo) (H) to part 2 (pelvis + feet) (B) and fix with 2 star screws (E).

Join the two parts just assembled together in the central part.
Secure everything with 2 star screws per side (2 screws on the right and 2 screws on the left) (E).


Insert the control unit (C)

Below the base of the bed has a housing for the control unit.


On the control unit itself, at the ends, there are two logos: "Head" and "Feet", to understand the direction.

("Head" direction towards the headboard and "Feet" direction towards the footboard)

Hook the control unit with a push from the bottom up.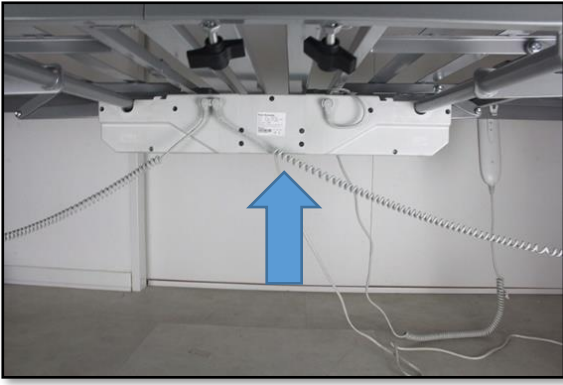


Lock it by inserting the 2 locking supports (D), they must slide sideways until they reach the limit stop.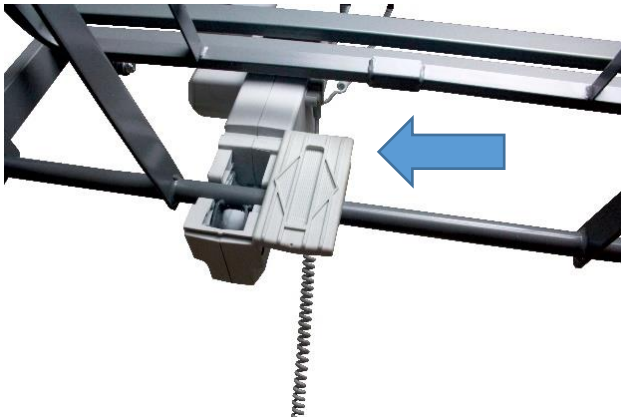


Hook the piston wires (located on the footboard and headboard) to the control unit.

Insert the sides (I):

- Raise the base of the bed about 1/4 to facilitate the insertion of the sides.
- At the ends of the sides, two by two, insert the guides (L) (an "upward arrow" logo gives the direction).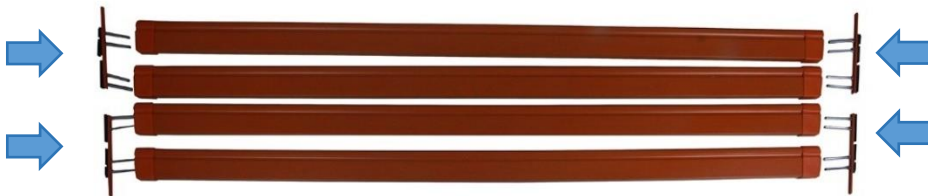

- Insert the sides from the bottom upwards until they click into the stop position.
- Fix the black screw (M) in the lower part of the guide for the sides. This safety system prevents the sides from coming out of their housing.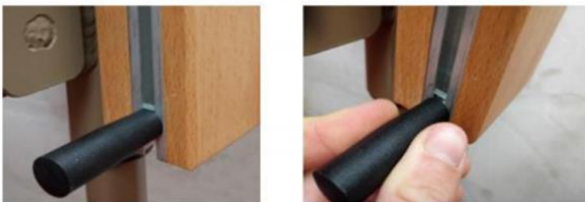

IMPORTANT! BEFORE USE


Connect the plug of the control unit to the electric socket, in this way the remote control will be active. Before use, check that all buttons are functional.

Attach the U-shaped support and Handle:

Anchor the handle (O) to the U-shaped support (N)

Insert the U-shaped support into the appropriate slot in part 1 (A).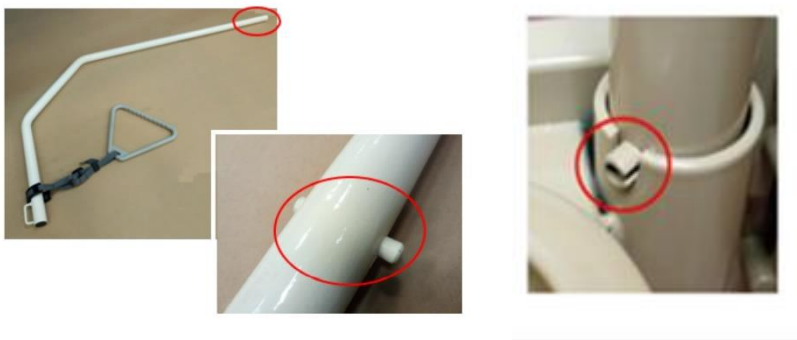

TROUBLESHOOT

INCONVENIENCE	SOLUTION
The bed doesn't answer to the UP/DOWN command	<ul style="list-style-type: none">- Check the plug is correctly plugged in- Repeat the activation process of the remote- Verify and clean the contacts- Reduce the load
The LED command of the remote is not working	<ul style="list-style-type: none">- Replace the LED
The battery does not charge properly	<ul style="list-style-type: none">- Contact the technical support service
The device moves intermittently	<ul style="list-style-type: none">- Remove the movement connectors from the remote and clean the contacts with a dry alcohol pad- Contact technical support
The panels are not properly inserted	<ul style="list-style-type: none">- Lubricate the machine- Contact technical support
Unusual sounds when the device is operating	<ul style="list-style-type: none">- Contact technical support

MAINTENANCE

This product requires regular maintenance.

If you have any doubts or problems do not hesitate to contact your dealer for assistance.

Warning! Keep the bed in a dry environment. Water and moisture could cause corrosion of electronic components and the chassis could rust.

If the product is exposed to water, perform the following maintenance:

1. Dry the water as much as possible with a rag and store it in a dry place;
2. Perform all safety and operational tests before use;
3. If there is any abnormality, please contact the dealer of this product.

WARNINGS

1. Avoid bumping in the electrical parts of the bed;
2. Check all electrical connections to make sure they are connected and securely fastened;
3. Check that all wheel bearings are lubricated and sealed.

PRODUCT INSPECTION

Daily Inspection :

1. Visually inspect the bed for external damage or wear;
2. Check if the screws and nuts on the boom, electric push rod and base are tight;

Monthly inspection:

1. Check the wear of the universal wheel. If maintenance is required, please contact your dealer.
2. Check the four wheels for wear and looseness, which may indicate that adjustments are needed or the bearings need to be replaced. Please contact the dealer for repair or replacement;
3. Keep the product clean and free of debris, such as hair, food, beverages, residues, etc.

STORAGE

This product should be stored in a dry environment with a storage temperature of $>0^{\circ}$ and a relative humidity of $\leq 60\%$. Do not place it at the extreme temperature.

CLEANING

Never rinse the product with water or contact it directly with water.

This product can be easily wiped clean with a damp cloth. Never use chemicals to clean the bed. This may cause corrosion. Clean it with a wet towel and a mild soap solution until it is completely dry.

GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.