MOPEDIA®

MA R1821_822_823_824_825_826_827_828 01 A_ITA_07-2021

SOLLEVATORI MUEVO

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

MOPEDIA®

MA R1821_R1822_R1823_R1825_R1826_R1827_R1828 01 A_ENG_07-2021

MUEVO LIFT

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7.1 Dimensions of structure with payload 150 Kg	pag. 5
7.2 Dimensions of structure with payload 180/200 Kg	pag. 5
8. DESCRIPTION OF THE PARTS	PAG.6
9. DESCRIPTION OF THE ACTIVATION KITS.....	PAG.6
9.1 Hydraulic actuator (150/180 Kg)	pag. 6
9.2 Electrical kit (150/200 Kg).....	pag. 6
10.ASSEMBLY INSTRUCTIONS FOR LIFT STRUCTURES (FOR ALL THE MODELS).....	PAG.6
11. ASSEMBLY INSTRUCTIONS ACTUATORS	PAG.7
11.1 Assembly instructions for hydraulic actuator (suitable for R1821-R1823-R1825-R1827)	pag. 7
11.2 Assembly instructions for electrical (suitable for R1822-R1824-R1826-R1828)	pag. 8
12. INSTRUCTIONS FOR USE	PAG.9
12.1 Instructions for using HYDRAULIC models R1821-R1823-R1825-R1827	pag. 9
12.2 Instructions for using electrical models R1822-R1824-R1826-R1828	pag. 9
13. WARNINGS	PAG.11
14. CLEANING	PAG.11
15. MAINTENANCE	PAG.11
16. SPARE PARTS AND ACCESSORIES.....	PAG.12
17. CONDITIONS OF DISPOSAL	PAG.12
17.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE	pag. 12
17.2 Treatment of dead batteries (Directive 2006/66/CE)	pag. 12
18. WARRANTY	PAG.12
18.1 Warranty for rechargeable batteries (if provided)	pag. 13
18.2 Reparations	pag. 13
18.3 Spare parts	pag. 13
18.4 Exoneration clauses	pag. 13

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

HYDRAULIC LIFTS

- RI821** HYDRAULIC actuator - payload 150 Kg - with pedal for legs opening
RI823 HYDRAULIC actuator - payload 150 Kg - with lever for legs opening
RI825 HYDRAULIC actuator - payload 180 Kg - with pedal for legs opening
RI827 HYDRAULIC actuator - payload 180 Kg - with lever for legs opening

ELECTRICAL LIFTS

- RI822** ELECTRIC actuator - payload 150 Kg - with pedal for legs opening
RI824 ELECTRIC actuator - payload 150 Kg - with lever for legs opening
RI826 ELECTRIC actuator - payload 200 Kg - with pedal for legs opening
RI828 ELECTRIC actuator - payload 200 Kg - with lever for legs opening

2. INTRODUCTION

Thank you for choosing a patient lift/stand up lift from the **MUEVO by MOPEDIA** product range by MORETTI S.p.A.. Their design and quality are a guarantee of comfort, safety and reliability. **MUEVO by MOPEDIA** patient lifts/stand up lifts by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the patient lift/stand up lift. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

MUEVO by MOPEDIA patient lifts/stand up lifts are intended to lift and move people with motor difficulties. An assistant operates the machine. The lifts cannot be used by only one person. MORETTI S.p.A. refuses all liability for damages caused by improper use of the device or by its use for a purpose other than that one specified in this manual.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PATIENT LIFTS MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)

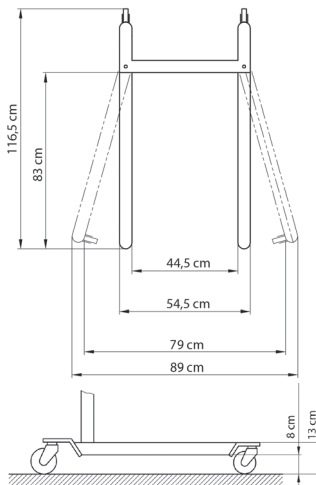
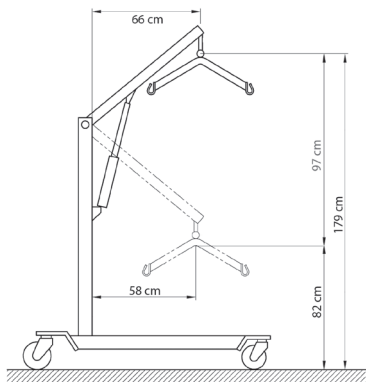
Protection against dust and water splashes

B Type

7. DIMENSIONS



7.1 Dimensions of structure with payload 150 Kg (RI821-RI822-RI823-RI824)

Total weight of 150 Kg lift structure with electrical kit included: **35 Kg**


Total weight of 150 Kg lift structure with hydraulic kit included: **33 Kg**

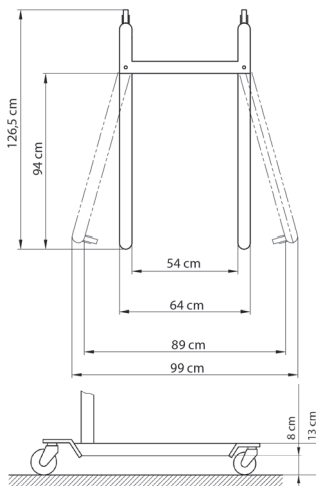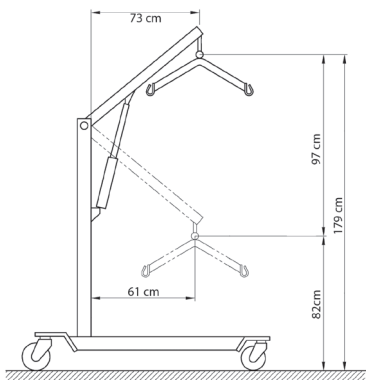7.2 Dimensions of structure with payload 180/200 Kg (RI825-RI826-RI827-RI828)

Total weight of 200 Kg lift structure with electrical kit included: **36 Kg**

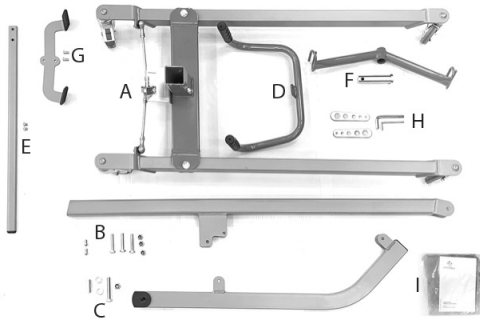Total weight of 180 Kg lift structure with hydraulic kit included: **34 Kg**

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
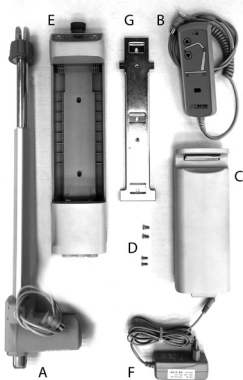
(image for illustrative purpose only)

10. ASSEMBLY INSTRUCTIONS FOR LIFT STRUCTURES (FOR ALL THE MODELS)

1. Unscrew the screw at the base of the column. (Fig.4)
2. Fit the column into the provided cavity of the base. (Fig.4)
3. Fit the screw that blocks the column to the base and screw it well. (Fig.4)
4. Insert the bushing in the arm hole arranged for the joint fixing with column (Fig.5a)
4. Connect and Fix the joint between column and arm with the provided screws (Fig.5b)

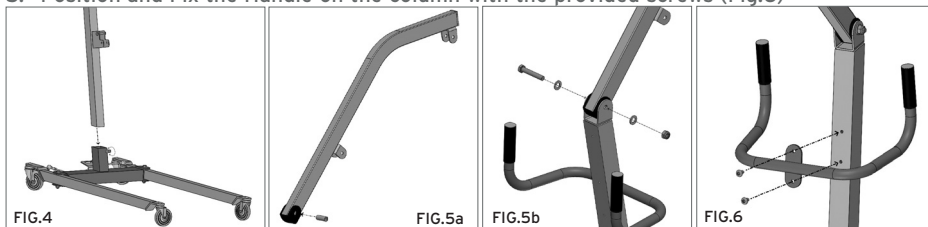5. Position and Fix the Handle on the column with the provided screws (Fig.6)

(image for illustrative purpose only)

6. Position and fix the slingbar with the provided screw on the corresponding holed bracket (Fig.7)


7a. Fit the pedal for base legs opening and block it with the provided screw.

(Fig.8a -for R1823-R1824-R1827-R1828 model)

7a. Fit the lever for base legs opening and block it with the provided screw.

(Fig.8b - for R1821-R1822-R1825-R1826 model)

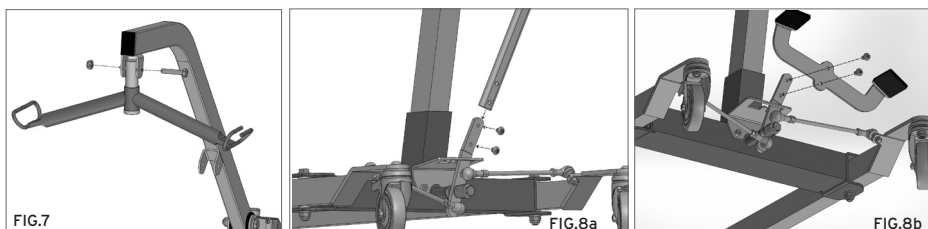
8. The structure will be completely assembled. Now install the electrical or hydraulic kit for lifting suitable for the model chosen.

(image for illustrative purpose only)

11. ASSEMBLY INSTRUCTIONS ACTUATORS

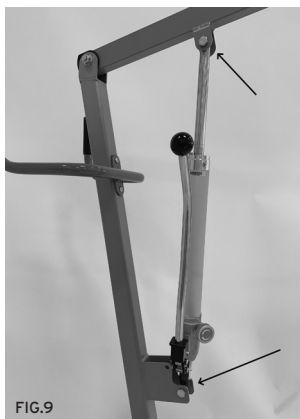11.1 Assembly instructions for hydraulic actuator (suitable for R1821-R1823-R1825-R1827)

(image for illustrative purpose only)

1. Place the hydraulic actuator as in the picture, blocking well its upper and lower part by auto-blocking screws and bolts. (Fig.9)

11.2 Assembly instructions for electrical (suitable for RI822-RI824-RI826-RI828)

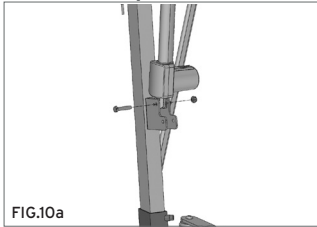
FIG.10a

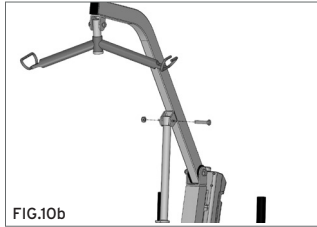
FIG.10b

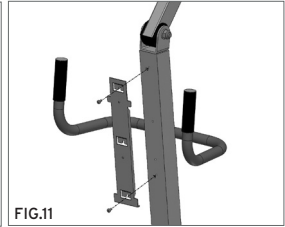
FIG.11

(image for illustrative purpose only)

1. Remove the bolts prepared from the arm and from the column holes for the actuator fixing. Fit the actuator in the correct position and fix it using the bolts previously removed. (Fig.10a-10b)
2. Fix the support bracket of the control unit to the column with the provided screws, screwed up the correct fitted holes. (Fig.11)
3. Fit the control unit on the bracket, letting it slide from top to bottom and blocking it with the provided screws. (Fig.12a-12b)
4. Fit the battery into the control unit cavity.(Fig.13a-13b)
5. Connect the actuator connector to the central socket of the control unit and the remote control to the right socket. (Fig.14). The electric kit is correctly mounted and the stand-up lift is ready to use. (fig.15)

FIG.12a

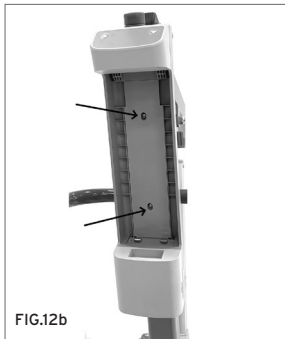
FIG.12b

FIG.13a

FIG.13b

FIG.14

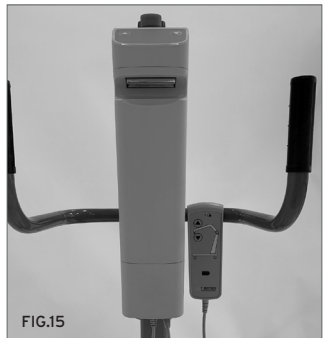
FIG.15

(image for illustrative purpose only)

Nota: Before use, be careful to check that the emergency button is not pressed, and if activated, release it to allow the switch on of the control unit.

12. INSTRUCTIONS FOR USE

12.1 Instructions for using HYDRAULIC models R1821-R1823-R1825-R1827

1. Before lifting a patient, open the base legs to the maximum width possible:
 - slightly pulling the pedal for legs opening towards you with foot, then rotating it to left and locking it in the open position (R1821-R1825)
 - slightly pulling the lever for legs opening towards you, then rotating it to left and locking it in the open position (R1823-R1827) (Carry out the same operations in reverse order to bring the base back to the closed status)
2. Block the rear wheel brakes
3. Keep the patient centred between the base legs and always turned towards the assistant
4. Hook the cradle straps securely to the lifting beam
5. Make sure the hydraulic pump valve is fully closed (turn it in the direction marked as CLOSE)
6. Activate the pumping lever to start lifting
7. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!
8. Lift the patient to the desired height, keeping him facing his assistant at all times
9. Release the rear wheels and move the patient
10. Before lowering the patient, block the rear wheels again
11. Start lowering the patient by slowly turning the hydraulic pump valve in the direction marked as OPEN

Note: To lower the patient slowly, open the valve just a little bit; opening it further always in the direction marked as "OPEN", if you want to lower the patient more rapidly

12. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

ALWAYS HELP THE PATIENT DURING THE LOWERING PROCESS SO AS TO MAINTAIN THE CORRECT SEATED POSITION!

WARNING!

While lowering the patient, please remove any barrier that can impede the downward movement. If the case, lift up the patient again and remove what causes the impediment for lowering.

12.2 Instructions for using electrical models R1822-R1824-R1826-R1828

BEFORE USING THE ELECTRICAL LIFT YOU ARE RECOMMENDED TO CHARGE THE BATTERY FOR ABOUT 6/8 HOURS. CONSEQUENTLY, MAKE YOU SURE THAT THE BATTERY IS CHARGED BEFORE THE USE

1. Before lifting a patient, open the base legs to the maximum width possible:
 - slightly pulling the pedal for legs opening towards you with foot, then rotating it to left and locking it in the open position (R1822-R1826)
 - slightly pulling the lever for legs opening towards you, then rotating it to left and locking it in the open position (R1824-R1828)(Carry out the same operations in reverse order to bring the base back to the closed status)
2. Block the rear wheel brakes
3. Keep the patient centred between the base legs and always turned towards the assistant
4. Hook the cradle straps securely to the lifting beam
5. Press the up-facing arrow key in order to lift the patient
6. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!
7. Lift the patient to the desired height, keeping him facing his assistant at all times
8. Release the rear wheels and move the patient
9. Before lowering the patient, block the rear wheels again
10. In order to lower the patient, press the down-facing arrow key
11. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

ALWAYS HELP THE PATIENT DURING THE LOWERING PROCESS SO AS TO MAINTAIN THE CORRECT SEATED POSITION!

WARNING!

While lowering the patient, please remove any barrier that can impede the downward movement. If the case, lift up the patient again and remove what causes the impediment for lowering.

AUTOMATIC ANTI-CRUSHING SYSTEM

The electrical kit is equipped with an automatic anti-crushing system, which stops the lowering of the patient lift if there is an accidental contact between the patient and any barrier.

EMERGENCY STOP

The control unit is equipped with a red “STOP” button which instantly deactivates all controls when pressed. To restore normal operation, turn the button clockwise and pull it outwards.

EMERGENCY KEYS

On the control unit there are two keys (one with an up-facing arrow and the other one with a down-facing arrow) that can be pressed instead of those ones on the remote control to

complete all the operations when the remote control fails to work properly.

MECHANICAL LOWERING OF THE ACTUATOR

If using the manual/mechanical lowering, please make sure that it will be done in total safety conditions for the patient. In order to activate the manual/mechanical lowering, pull downwards the red ring placed in the upper part of the actuator and turn it counter-clockwise. Once finished the lowering process, take the red ring back to its resting position.

INDICATION OF THE BATTERY STATE OF CHARGE

There is a 3-Led indicator for the battery state of charge. This indicator is placed on the remote control. 3 Leds on indicate a complete charge of the battery, whereas one just Led on indicates the battery is out of power and needs to be charged. The standard battery, when new, with full charge and outside temperature of use and storage at 20 ° C, has an autonomy of over 40 cycles with a load of 200 kg; when the lifted load decreases, there is an increase in the autonomy of the battery and therefore of the cycles feasible.

RECHARGING BATTERIES

- Connect the adapter to the main socket and then to the socket on the control unit, after removing the protection cap
- On the charger adapter there is a led that during the charging process becomes orange color. Charge the battery until the led becomes green color.
- Before activating the lift again, make sure to disconnect the battery charger!
- Charge the batteries for at least 6 hours before using the lift

INSTRUCTIONS FOR USE OF REMOTE CONTROL

The remote control is connected to the control unit by a spiral extendable cable

Keys:

1. UP" key for lifting
2. DOWN" key for lowering

Visual indications:

3. LED for max. load exceeded: the lift movement is interrupted if the load exceeds the max. load threshold and the control unit generates an acoustic signal.
4. Led for monitoring and signaling the battery charge status

13. WARNINGS

- The lift is a device designed for moving patients with motor problems for short distances, for instance from a bed to a wheelchair or comfortable chair, and back. It must not be used for transporting the patient.
- While lifting the patient, make sure the base legs are as far apart as possible, the control lever is securely locked and the rear wheel brakes are secured.
- While moving the patient, please avoid crossing barriers that might cause the lift to tip over.
- The use of the patient lifts described in this manual, in combination with the Moretti harnesses provided among the accessories, it allows to pick up and lift patients off the ground.

WARNING!

Always check that the cradle is in good condition. Do not use cradles showing signs of damage or wear.

14. CLEANING

To clean the cradle and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

15. MAINTENANCE

- It is extremely important to check the lift before each use.
- More specifically, please check that all screws and bolts are well tightened.
- Please, check the lift can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.
- It is very important to regularly check the connection pins and stop rings are correctly located (the pins connect the actuator to the upper and lower part of the lift, and the lifting beam where you hook the cradle).
- The mechanical parts must be lubricated with a drop of oil in order to facilitate their sliding and resistance to wear. In particular, pay attention to the mechanical retractor for the legs of the patient lift. Lubricate as indicated above every two or three months.

16. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, please always refer to the MORETTI master catalogue

Accessories

Cod.RP828 Stretcher structure for patient lifts (max load 150 Kg)

Cod.BLD500-BLC250 Scale for patient lifts Class III

Cod.RP881 Standard harness with 'basic' headrest

Cod.IM100-IM101 Standard harness without headrest

Cod.IM110-IM111 Standard harness with headrest

Cod.IM120 Containment harness with splints

Cod.IM140 Toilet harness without headrest

Cod.IM141 Toilet harness with headrest

Cod.IM150 Containment harness for amputees

ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

17. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

17.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

17.2 Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

18. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery)

or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18.1 Warranty for rechargeable batteries (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

18.2 Reparatons

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

18.3 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

18.4 Exoneratation clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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