Break the inside water bubble, by squeezing the bag with both hands.
Premere con entrambe le mani verso il centro fino ad ottenere la rottura della bolla interna.
Casser la bulle à l'intérieur du sachet en pressant avec force des deux mains.
Presionar con las dos manos hacia el centro hasta obtener la rotura de la burbuja interna.

Then shake it in order to mix the contents.
Agitare il sacchetto per miscelare il contenuto.
Ensuite agiter le sachet pour mélanger le contenu.
Agitar la bolsa para mezclar el contenido.

Apply on the interested part.
Applicare sulla parte interessata.
Appliquer sur la partie intéressée.
Aplicar sobre la parte interesada.

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INSTRUCTIONS FOR USE: break the inside water bubble, by squeezing the bag with both hands. Then shake it in order to mix the contents. Apply on the interested part.
INDICATIONS: haematomas, sprains, injuries, hyperthermia and whenever cold application is required.
CAUTION: for external use only; disposable product. Do not use if the bag is damaged. Do not break the bag and by all means prevent any content leakage. The content of the bag causes stains and irritation (in case of contact, rinse with water for 15 minutes), and is harmful (if swallowed seek immediate medical assistance). Store at room temperature out of the reach of children and away from heat sources and flammable materials. Do not use on injured skin. Contact your doctor in case of cardiovascular diseases, circulatory diseases, diabetes and/or cutaneous insensitivity (Raynaud's syndrome), if the product is used on infants, children and the elderly, or in case of doubt on how to perform the treatment and if the symptoms persist.
CONTENT: urea, ammonium chloride and water.

Instant cold compress Ghiaccio istantaneo in busta TNT

- Pochette de froid instantané
- Hielo instantáneo (frio)
- Sofort-Kälte-Kompresse (Kalt)
- Gêlo instantâneo (frio)
- Στιγμιαίος πάγος (κρύος)
- تلج فوري (بارد)

REF **34110** 14x18 cm - TNT bag

 **Gima S.p.A.**
Via Marconi, 1
20060 Gessate (MI) - Italy
Made in Italy

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