MOPEDIA®

MA MP180_MP181_02 A_ITA_07 2020

SPONDE RIBALTABILI

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ITALIANO

INDICE

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CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

MP180 Anodized aluminum folding bedside rails - pair

MP181 Painted steel folding bedside rails - pair

2. INTRODUCTION

Thank you for choosing an accessory MOPEDIA by Moretti. The bedside rails have been designed and built to meet all your needs for a practical correct use, correctly and safely. This manual contains some suggestions for proper use of the accessory and valuable tips for your safety. You should read this manual carefully before using the bedside rails. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The bedside rail is an accessory designed to prevent an accidental fall of a person from a any bed for hospital or home care.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ACCESSORIES FOR BEDS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

CAUTION!

- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Always pay attention to the presence of children.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

WARNING!

- This device must not be used as support for patient transfers.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

The bedside rails are designed to fit beds thank to its simple frame to be placed on the headboard and the footboard.

7.1 Parts List

MP180 - MP181

Bedside rails assembled (2 pcs)

8. INSTALLATION/REMOVAL

8.1 Assembly MP180 - MP181

The bedside rail are completely assembled, and installation is quick and easy, as in the example above.

1. Place the hooks on one side against the headboard and extend hooking the two hooks on the opposite end.
2. Repeat from the opposite side.

8.2 Dismantling MP180 - MP181

Follow the procedure described above in reverse.

9. BEFORE USE

- Always check the wear of the mechanical parts in question to ensure use of the device in total safety for persons and things.
- Check carefully the proper assembly of the device and pay attention fixing the bedside rails on the bed frame.

10. MAINTENANCE

The MOPEDIA devices by Moretti S.P.A. are checked carefully and supplied with **CE** mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use only a damp cloth and mild soap. Afterwards dry thoroughly before using

11.2 Disinfection

If you need to disinfect the product use a common disinfectant.

Note: Never use acids, alkalines or solvents such as acetone or thinner.

12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the thermometer in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES

MP180 Anodized aluminum folding bedside rails - pair

MP181 Painted steel folding bedside rails - pair

13.1 Dimensions

- Useful bedside rail length 170 cm
- Overall length 195 cm
- Max extension length 222 cm
- Max coupling length 210 cm
- Tube Ø2,5 cm
- Height of side 43cm
- Distance between 2 hooks 26 cm

13.2 Technical specifications

- Anodized aluminum frame MP180
- Painted steel frame MP181
- Hooks and fittings in ABS

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15. REPAIRING

15.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

15.2 Repair not cover by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

15.3 Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

16. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

17. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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